


Verify ► Bend ► Fit ► Trim

CONSULT ALL APPLICABLE INSTRUCTIONS FOR USE PRIOR TO SURGERY


**Step 1: Verify Hardware**

- i. Confirm that the work order number and patient name on the Mandible Model are correct.
- ii. Confirm Stryker Stock plate matches the specifications indicated in the Case Report.

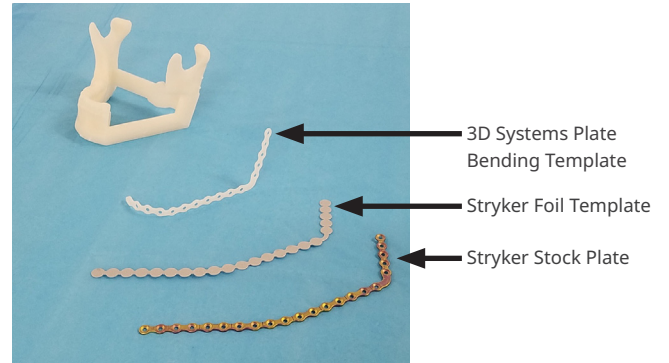


**Step 2: Bend Plate**

- i. Do not trim any holes before bending the Stryker Stock Plate.
- ii. Place and bend the Stryker Foil Template to the inside of the 3D Systems Plate Bending Template.
- iii. Then place the Stryker Foil Template on the inner surface of the Stryker Stock Plate, and bend the Stryker Stock Plate.

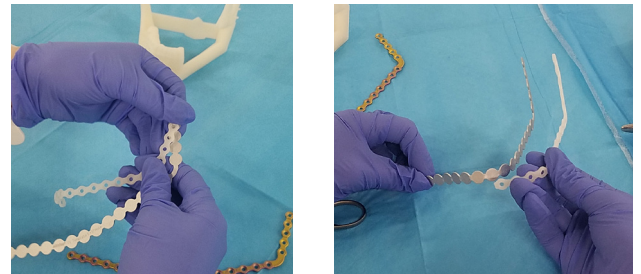

**Step 3: Test Fit Plate to Model**

- i. Test fit the Stryker Stock Plate with the registration features on the provided model.

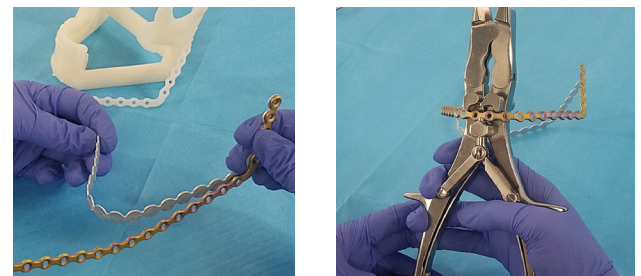

**Step 4: Trim Plate**

- i. Trim the excess holes on the Stryker Stock Plate using Stryker instrumentation.
  - ii. Sterilize the Stryker Stock Plate prior to use in accordance with Stryker IFU.
- Note: Refer to the Case Report for excess plate hole removal.*

