


Place ▶ Mark ▶ Osteotomize ▶ Align
CONSULT ALL APPLICABLE INSTRUCTIONS FOR USE PRIOR TO SURGERY

Step 1: Place Cranial Marking Guide


- i. Confirm that the work order number and patient name on the Cranial Marking and Positioning Guides are correct.
- ii. Ensure that the tissue is removed from the bony surface.
- iii. Place the Cranial Marking Guide and ensure that the guide is contacting the bony surface.

Note: Refer to the Case Report for patient-specific guide position.

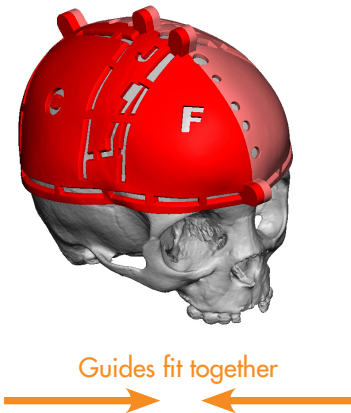


Step 2: Mark Osteotomies

- i. Mark the osteotomies along the indicated path.
- ii. Mark the registration letters along the indicated path, if applicable.
- iii. Remove the Cranial Marking Guide.

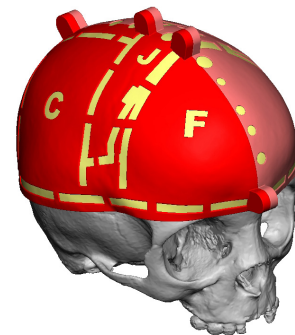

Step 3: Perform Osteotomies

- i. Perform the osteotomies along the marked locations.

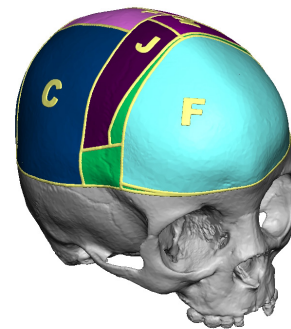

Step 4: Align with Cranial Positioning Guides

- i. Place the Cranial Positioning Guides on a flat and stable surface.
- ii. Align the marked registration letters on the bony segments to the letters on the Cranial Positioning Guides for correct positioning.
- iii. Plate internally or externally, if applicable.

